IncoStress Instructions For Use

IncoStress is latex free product and has been clinically approved to provide positive results to the majority of adult women who suffer with urinary stress incontinence.

Intended purpose of IncoStress IncoStress is designed for women who wish to control the involuntary loss of urine and/or who wish to improve their pelvic tone. Note: IncoStress® is a class IIa medical device. Any incontinence issues should be discussed with your doctor.

How it works IncoStress is worn the same method as a tampon, it supports and restores the natural anatomical position of the bladder neck the urethra (the tube that allows loss of urine from the bladder). IncoStress controls stress incontinence and can also be used as a bladder training device in the case of over active bladder and used as a biofeedback device. Its unique shape enables you to identify the pelvic floor muscles allowing an isometric contraction so you can carry out kegel exercises.

Inserting IncoStress is worn in the same way as a tampon. It supports the urethra and bladder neck which immediately controls the involuntary loss of urine. Its unique shape also allows the correct pelvic floor muscles to be identified and exercised. Regular use and exercise is recommended to help strengthen them over time.

Before you insert IncoStress

- Empty your bladder and wash your hands.
- Remove IncoStress from the packaging and wash under clean water.
- Normally you should not need lubrication as your body produces this naturally. If you should need extra lubrication we recommend water based on the top end of IncoStress.

Picture 1: Insert IncoStress into the vagina (thick part first) as high as it will allow. In some cases the thin tail will be completely inside the vagina, it will always be within easy reach to remove. You may feel the presence of IncoStress, but it should not be uncomfortable. However, if you do feel any discomfort, remove, wash and re-insert.

Picture 2: IncoStress in position, demonstrating the support of the bladder and urethra.

Removing Relax the pelvic muscles, gently pull out IncoStress using the tail. If you find it difficult to retain IncoStress, there may be other medical conditions which contribute or it may be that your pelvic muscles are too weak to hold IncoStress. In this case we recommend you use IncoStress during sleeping hours as your pelvic muscles can work whilst asleep, helping them regain strength naturally over a period of time. If you have any doubts, we strongly recommend you seek professional medical advice.

Usage IncoStress is to be removed, cleaned and inspected daily. It may

be used up to 6 months after the package is initially opened. C&G Medicare Ltd will not take any responsibility if this product has been used outside recommendations contained in these instructions. IncoStress may be used during sleep. We recommend a maximum of 8 hours continuous use. After 8 hours remove, wash and leave out of the body for a minimum of 1 hour. IncoStress may be removed as many times as you feel necessary during the 8 hours period. It is recommended that IncoStress be removed before one opens the bowels and before urinating. Those with strong muscle control may find they won't need to remove it during toileting.

Cleaning, Inspection and Replacement Visually inspect for any signs of discoloration, wear or damage daily. IncoStress should be replaced every 6 months after initial opening of the package. Do not clean using alcohol, bleach or any other chemicals. Do not place in the dishwasher. Do not place in the microwave. Do not steam.

Disposal IncoStress can be disposed of by simply wrapping in tissue or toilet paper and disposing with normal household waste. It must not be flushed down the toilet as it is not biodegradable and may cause blockages.

Warning and Contraindications IncoStress is made from body friendly Medical Grade Silicone. Should you find you have any sensitivity to this material or are allergic, you must stop using the product immediately and consult your doctor. Toxic Shock Syndrome (TSS) is a rare but a serious illness, it can be treated if recognised early. TSS has never been linked to medical grade silicone pessaries. For further details regarding Toxic Shock Syndrome please visit www.incostress.com

Symptoms of TSS Sudden high fever (usually over 39°C/102°F), vomiting, diarrhoea, sunburn like rash, sore throat, dizziness or fainting. Not all these symptoms may be present at once. In the unlikely event that you have these symptoms during use of the IncoStress, remove it immediately and consult your doctor. There have never been any reported cases of TSS with the use of IncoStress.

Important IncoStress has been researched and designed explicitly for the intended purpose as stated in these instructions for use, and must NOT be used outside of these instructions for any other purpose.

IncoStress is a SINGLE PATIENT USE product and must NOT be interchanged between patients.

IncoStress must NOT be left in place any longer than 8 hours without removal, cleaning and inspection. IncoStress must NOT be flushed down a toilet.

IncoStress must NOT be used during menstruation and never use a tampon at the same time.

IncoStress must NOT be used if you are or think you are pregnant.

DO NOT USE IncoStress if you have any urine, bladder or vaginal infections. DO NOT USE IncoStress if you have any known allergies to any sort of silicone.

Additional advice and problem solving on how to use your IncoStress®

- IncoStress should be gently pushed high up into the vagina, until the tail is still easily reached.
- If at first IncoStress is uncomfortable or tends to slide out, don't worry as this sometimes happens. Your vagina is not familiar with holding this type of shaped device in place and needs to adapt itself. This is best done when lying down and you should then stay lying flat with legs partly drawn up for 15 to 30 minutes.
- During this time try relaxing then contracting your pelvic floor muscles whilst supporting the IncoStress with a finger tip if necessary.
- When you tighten the muscles the IncoStress should tend to move away from your finger tip. If it comes lower down you may be doing the wrong exercise and pushing instead of lifting up.
- Keep practising until the muscles are working properly. Do not work the muscles too hard on your first day as they may become very tired; this is not necessary and may become uncomfortable.
- Repeat this exercises several times over the next few weeks until you are used to hold it in place. You are then ready to begin to walk around with your IncoStress in position. For some women it will hold its position very well. If it still slips out then try simply standing and tighten your pelvic muscles. If you do this properly it should lift upwards. KEEP ON PRACTICING. The more you do this, the stronger your pelvic floor will become, and more you will control your bladder.
- Some patients find that wearing IncoStress during the night is the best way of getting used to its special shape and concept.
- A lot of work and research has gone into the design of IncoStress, and it is important that that could help you. For some women suffering, prolapse or other acute or chronic problems IncoStress may not stay in position. If this is the case, we advise you contact your doctor or specialist. Don't discard IncoStress.

Keep it safe, as it may be very useful after physiotherapy or surgery.

Download the kegel exercises from our website
C&G Medicare Ltd is constantly aware of keeping our customers satisfied.
www.incostress.com Email: info@incostress.com Tel: 0871 218 1233
UK Registered office: 44 Victoria Gardens, Neath, SA11 3BH
Returns to: C&G Medicare Ltd, Williams House, 11-15 Columbus Walk, off Bringantine Place, Cardiff CF10 4BY

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